

INFORMATION FOR USE FOR EASY SURGICAL SETS

Validity Range

For all EASY SURGICAL series drape sets.


Designation

Information on using EASY SURGICAL drape sets.


Description

This information sheet is for giving instructions on usage of EASY SURGICAL drape sets.

How to Use

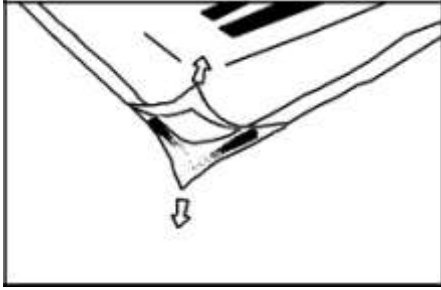

Open package.

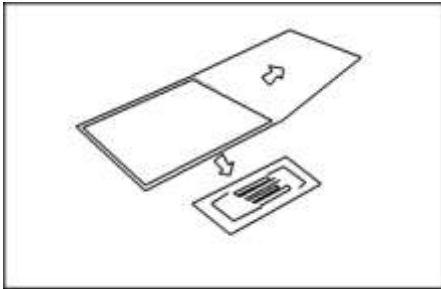

Take out pouch & gloves.

Open pouch.

**** Pouch is an optional feature. If there is no pouch use items in given order*



Place drapes in an appropriate manner to area in accordance with procedure.
Use other content according to procedure.

Symbols Used



Date of Manufacture



Expiration Date



Lot Number



Referance Number



Ethylene oxide Sterlization

Shelf Life

Product has been tested for stability. Shelf life of each product can be seen on the package of the product.

Storage

Store the product under normal conditions for healthcare material, in a cool and dry place protected from light. The use of medical grade paper and film guarantees its sterility for the duration of its shelf life under normal transport and storage conditions. Please store in dust free areas.

Disposal

Products contain fabric and/or plastic and/or metallic components. These components should be evaluated as medical waste and treated according to national and regional laws and regulations.

Comments

The product must be used by specialised personnel only. Check the package's integrity before use. Do not use if the package is open or damaged.